
**Glitter Gumball Switches #7830B, G, M,
P & Y**

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Add some glitz to your life!

We've jazzed up our most popular Gumball Switches by adding a variety of colored glitter to the tops. Seven different colors to choose from or buy all seven for your classroom. Our high quality, dependable switch provides both audible and tactile feedback. Comes with permanent clear plastic covers to protect the glitter surface. Base has three holes for easy mounting. Mounting plate is removable. Size: 3"D x 3/4"H. Weight: 1/4 lb.

Operation:

1. The Glitter Gumball Switch does **not** require any batteries to operate.
2. Plug the Glitter Gumball Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Glitter Gumball Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Glitter Gumball Switch's top to activate your toy/device.
6. The toy/device will only remain activated while the Glitter Gumball Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Mounting Plate Options:

The Glitter Gumball Switch has a removable mounting base option. To use without the base, simply remove the two small Phillips Head screws holding the base from the bottom of the switch body

Please Note: Clear Top is Not Removable.

Troubleshooting:

Problem: The Glitter Gumball Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Glitter Gumball Switch and your toy/device is tight. There should be **no** gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Glitter Gumball Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.