

---

---

# Mini Gum Ball Switches #783R, B, G, Y, P

---

---

## USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

The Mini-Gumball switch is versatile and durable. It has a 1¼ inch activation surface. This switch is small enough to be easily mounted anywhere. Base has three holes for easy mounting. It provides tactile and audible feedback. Mounting plate is removable. Size: 2"Base x ½"H. Weight: 1 oz.

### **Operation:**

1. The Mini Gum Ball Switch does **not** require any batteries to operate. The base is removable if desired.
2. Plug the Mini Gum Ball Switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Mini Gum Ball Switch and your toy/device is plugged in all the way. There should be no gaps.
6. Simply press anywhere on the Mini Gum Ball Switch's colored top to activate your toy/device.
7. The toy/device will only remain activated while the Mini Gum Ball Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

### **Troubleshooting:**

**Problem:** The Mini Gum Ball Switch does not activate your toy/device.

**Action #1:** Make sure that the connection between the Mini Gum Ball Switch and your toy/device is tight. There should be **no** gaps. This is a common error and an easy fix.

**Action #2:** Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

**Action #3:** Try a different adapter (if applicable) to rule this out as the source of the problem.

### **Care of Unit:**

The Gum Ball Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.