
Mini Gum Ball Switches #783R, B, G, Y, P

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

The Mini-Gumball switch is versatile and durable. It has a 1¼ inch activation surface. This switch is small enough to be easily mounted anywhere. Base has three holes for easy mounting. It provides tactile and audible feedback. Mounting plate is removable. Size: 2"Base x ½"H. Weight: 1 oz.

Operation:

1. The Mini Gum Ball Switch does *not* require any batteries to operate. The base is removable if desired. To remove the base turn the switch over and unscrew the two Phillips Head screws. Next pull the base ring off the switch. After the base has been removed you can place the included piece of non skid rubber, to give the switch a firm grip on smooth surfaces.
2. Plug the Mini Gum Ball Switch into the jack on your switch-adapted toy/device.
4. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
5. Make sure that the connection between the Mini Gum Ball Switch and your toy/device is tight. There should be no gaps.
6. Simply press anywhere on the Mini Gum Ball Switch's colored top to activate your toy/device.
7. The toy/device will only remain activated while the Mini Gum Ball Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Mini Gum Ball Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Mini Gum Ball Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Gum Ball Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.