
Thumb Switch # 802

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Capitalizes on thumb mobility!

Grip the yellow cylindrical base in your hand and press the red button with your thumb for momentary switch operation.

Size: 4½"L x 1"W

Weight: 3 oz.

Operation:

1. The Thumb Switch does **not** require any batteries to operate.
2. Plug the Thumb Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Thumb Switch and your toy/device is plugged in all the way. There should be no gaps.
5. Simply press the top of the Thumb Switch, to activate your toy/device.
6. The toy/device will only remain activated while the Thumb Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Thumb Switch does not activate your toy/device.

Action #1: Make sure that the Thumb Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

Action #2: Make sure that the connection between the Thumb Switch and your toy/device is plugged in all the way. There should be **no** gaps. This is a common error and an easy fix.

Action #3: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #4: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Thumb Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the switch.