
Adapted Llama Musical Stacker

Stack this toy's three colorful rings or press on the llama's head or switch (not included) to make lights dance and music play! This cute little llama plays more than 25 songs, tunes and phases. Learning to distinguish between sizes and colors has never been this rewarding! The Llama Musical Stacker is part of the Fisher Price Linkimals family and interacts with the *Smooth Moves Sloth #7579* ([link](#))! Size: 6½"D x 9½"H. Weight: 2 lbs. Battery: 3 AA

Helps with:

- Encouraging Reaching
- Developing Fine Motor Skills
- Increasing Visual Attention
- Teaching Cause & Effect

Operation:

1. The Llama Stacker requires 3AA batteries. The battery compartment is located under the base of the unit. To install batteries turn the stacker over. Next remove the small Phillips head screw and cover. Install batteries being careful to **observe proper (+) & (-) battery polarity**. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well. Set the Volume switch to the Low or High setting.

2. Stacking the rings in the correct order or using their switch, will reward the user by making lights dance and music play! This cute little llama plays more than 25 songs, tunes and phases. Learning to distinguish between sizes and colors has never been this rewarding!

Operation with a switch:

3. Connect your switch via the 1/8-inch jack on the unit. **Please Note:** the first time you plug your switch into the Llama Stacker it will play a song, this is normal do to switch adaptation.
4. Make sure that you have your switch plugged in all the way. There should be no gaps.
5. Now, when you activate your switch a song will play in its entirety then stop. After the song finishes if you press your switch again it will play a new song. There are a total of five songs all together. Songs will play in a random order.

Troubleshooting:

Problem: The Llama Stacker will not operate.

Action #1: Check batteries for proper installation and freshness. Replace batteries if weak or dead.

Action #2: Make sure that all connections between cords, adapters, and devices are tight.

Care of Unit:

The Llama Stacker can be wiped clean with any mild household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Adapted Llama Musical Stacker #8025

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com