



**enabling**  
devices

**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)

---

---

## Adjustable Angled Switch/Communicator #806

---

---

### USER'S GUIDE

**enabling**  
devices

50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### **Bright lights!**

Our sequencer is designed for the visually impaired. The circular patterned red LED lights in the center give the user a visual cue for targeting and encourage the user to activate the switch. The angle of the big 5" switch is adjustable from 32° to 90°. The light can be turned on or off. This sequencer has a total of 600 seconds recording time and can be clamped. Also works as a switch. Easy to mount on tabletops with a clamp. (C-clamp sold separately.) Size: 8"L x 8"W x 6¼"H. Requires 4 AA Batteries. Weight: 1¼ lbs.

#### **Operation:**

1. Adjustable Angled switch Communicator requires four AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Gently turn the unit over to reveal the battery compartment. Carefully remove the small screw from the battery compartment cover using a Phillips head screwdriver, and lift the cover off. Observing proper (+) & (-) battery polarity, install four AA size batteries into the holders. Replace the battery compartment cover and screw.
3. The LED lights ON/OFF SWITCH is above the battery compartment. When turned ON, the LED lights will stay ON until this switch is turned OFF. Please Note: when finished using the switch turn the LED's OFF to prevent battery drain. The second control is the ON/OFF/Volume knob. This is used when the switch will be used as a communicator.
4. To make a Recording, turn the ON/OFF/VOLUME knob ON you will hear a slight click, then adjust the volume to an audible level. **Please Note: when recording make sure the LED lights are turned OFF.** The **Microphone** is located on the left side of the unit. To record depress the RECORD button and speak 6 to 8 inches from the microphone. When you are finished recording the first message, release the RECORD button. To record all subsequent messages, simply depress the RECORD button, speak into the microphone, releasing the RECORD button after each message.
5. Messages can vary in length and the total record time is 600 seconds. To play the messages. Activation of the Yellow switch plate will play the first message; the second activation will play the second message, and so on. To interrupt the messages and start at the beginning of the sequence, turn the unit off and wait 2 to 3 seconds.
6. The unit will retain recorded information indefinitely, even if the batteries are removed. Previously stored recorded messages can be removed by doing a RESET. To perform a reset do the following.
7. With the unit OFF. Press and hold down the RECORD button while turning the ON/OFF/VOLUME knob ON while keeping the RECORD button depressed for 5 to 7 seconds (the unit will start to tick slowly) then release.

The unit will continue to tick faster. Wait until the ticking completely stops. Turn the unit OFF, wait 5 seconds then turn back ON. Now all previous messages will be erased from the device.

8. The Adjustable Angled switch can be adjusted from 20 to 90 degrees by simply losing the two knobs on the side of the unit and pulling up the big 5" Yellow Plate to the desired angle. Once you have the desired angle simply tighten the two set knobs to lock in the angle. Do not over tighten the set knobs. Finger tight is all that is needed. (See photo on back cover.)
9. All it takes is a slight push on the plate, a roll up action by your hand or a swiping action to activate the switch. There are also three different spots on the black base mold to clamp the switch to a table top or wheelchair tray.
10. To use as a switch, attach one end of the double ended 1/8" to 1/8" male cord (included) into the back of the Adjustable Angled switch, via the 1/8" jack. Next plug the other end of the cord into your adapted toy or device. Make sure all connections are tight with no gaps. When the yellow Plate is pressed the toy or device will activate. Release the plate and your toy/device will turn off.

#### **Troubleshooting:**

**Problem:** My toy/device does not work with the Adjustable Angled switch.

**Action #1:** Check the batteries for proper installation in your toy/device.

**Action #2:** Check batteries for freshness, if weak or dead replace with new batteries.

**Action #3:** Make sure there is nothing obstructing the yellow switch plate on the Adjustable switch.

**Action #4:** Make sure all connections between Adjustable Angled switch and your toy/device are tight. There should be no gaps. If using an adapter it must be a mono adapter, not stereo.

**Action #5:** Check your toy/device with another switch to rule the Adjustable Angled switch out as the source of the problem.

#### **Care of Unit:**

The Adjustable Angled switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the contents and the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.