
Baby Retriever #889

USER'S GUIDE



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-914-747-3070 or 1-800-832-8697

customer_support@enablingdevices.com

See pooch walk!

This adorable, lifelike little pup barks, wags his tail and swings his head when his human playmate activates an external capability switch (not included). A great cause and effect experience. Size: 6"L x 7"H. Requires Capability Switch and 2 AA Batteries. Weight: ½ lb.

Operation:

1. Baby Retriever requires 2 AA batteries (not included) to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn Baby Retriever upside down to reveal the battery compartment. Install 2 AA batteries, paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure it. Connect Baby Retriever to any external capability switch (not included) via the jack located at the end of Baby Retriever's cord. Baby Retriever is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to make a tight connection. There should be no gaps.
4. Actuate your external capability switch and Baby Retriever will spring to life! He will walk forward and bark.
5. Baby Retriever will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Baby Retriever will turn off.

Troubleshooting:

Problem: Baby Retriever does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit Mini Baby Retriever's movement, as this may cause irreversible damage.
- Do not lift Mini Baby Retriever up by his head, tail or legs, or drop Baby Retriever.
- If you will not be using Baby Retriever for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Baby Retriever is surface washable only.