

Care of Unit:

When you receive your mouth stick stylus you should sanitize the mouthpiece with at least 70% Isopropyl alcohol. We also recommend cleaning after daily use. The Adjustable Stylus mouth piece can be cleaned with 70% Isopropyl alcohol or hot water and dish washing detergent. Then allowed to air dry or pat dry with a clean cloth.

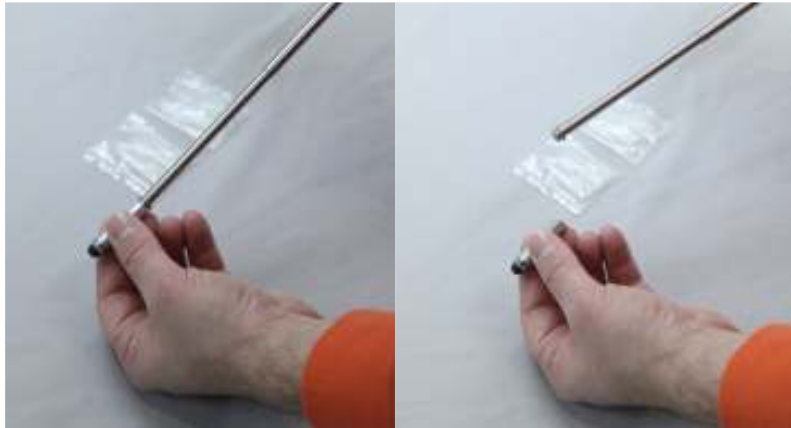


Fig.3



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

Adjustable Touch Screen Mouth Stylus #9089

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Sturdy yet lightweight!

This lightweight adjustable stylus works with the slightest touch on your tablet or smart phone. Its telescoping stainless steel shaft easily adjusts from 8½" to 15". The plastic bite plate with rubber guard allows the user to firmly yet comfortably hold the stylus. *Size: 3/8"Diameter x 8½" to 15"L. Weight: 1½ oz*

Operation:

1. When you receive your mouth stick stylus you should sanitize the mouthpiece with alcohol. To begin using you my shown in figure 1.



Fig.1

2. The Adjustable Stylus is on a telescopic rod you can adjust its length to fit your needs. The Adjustable Stylus telescoping stainless steel shaft easily adjusts from 8½" to 15". The mouth piece angle is also adjustable with the thumb screw and replaceable (special request) See figure 2.



Fig.2

3. Replacement Tips:

When it comes time to replace the Stylus tip you will first need to remove the band of silver tape wrapped around the base of the Stylus. Next you will unscrew the tip and replace it with one of the replacement tips (sold separately). See figure 3. On back page. The replacement tips will include a new tape strip to re-apply to secure the tip to the Stylus. **Please Note:** The tape must be used to prevent the tip from becoming a choking hazard for young children.