
MINI COM SEQUENCER #9151-S

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Great for building communication skills!

We've added sequencing to our best-selling Mini-Com! Great for storytelling, singing and joke telling.

- 300 seconds of recording time
- Attaches easily with each other
- Has icon, picture or word holder

Size: 5½"L x 6"W x 3"H. Requires 4 AA Batteries. Weight: ½ lb.

OPERATION:

1. Carefully turn over unit to reveal battery compartments. Install 4 AA batteries (Not included), observing proper polarity (+/-), in battery holders. Use a good brand of alkaline batteries; do not use rechargeable batteries because they do not provide enough voltage to power this device.
2. The **MICROPHONE, VOLUME ON/OFF** knob and the **RECORD/PLAY** switch are located on the back panel of the unit. First turn the unit on by turning the VOLUME knob in a clockwise direction. Make sure it is set to an audible level. To record, flip the **RECORD/PLAY** switch to the **RECORD** position. Depress the switch plate and speak the first message. When you are finished recording the first message, release the switch plate. **Please note do not depress the switch plate while turning the unit ON, doing so will erase any pre-recorded messages.** To record all subsequent messages, simply depress the switch plate, speak into the microphone, releasing the switch plate after each message.
3. Messages can vary in length and the total record time is 300 seconds.
4. To play the messages, flip the **RECORD/PLAY** switch to the **PLAY** position. Activation of the switch plate will play the first message; the second activation will play the second message, and so on. To interrupt the messages and start at the beginning of the sequence, press the **RESET** button on the back of the unit.

5. The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when new recording is performed.
6. **Please Note:** When re-recording messages, all messages on the unit will be erased.
7. To turn the unit off, turn the volume control knob counter-clockwise until you hear an audible "click"

IMPORTANT NOTES:

The memory of this unit is capable of holding up to 300 seconds of speech information total.

If the unit fails to operate, or performs incorrectly, the following steps should be performed:

- Disconnect the batteries for 30 seconds to reset the unit. If the unit still does not work after batteries are reconnected, replace the batteries with new ones.
- Make sure there are no obstacles between the switch plate and the housing of the unit.

If you have any problems or questions about this unit or any of our products, please call our Technical Assistance Department

Care of Unit:

The Mini Comm. Sequencer can be wiped clean with any mild household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.