

---

---

## Dance and Move BeatBo #9231

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### **Energizing music!**

Introduce letters, colors, counting, music, cause & effect and much more. Activate any one of the three large switches on the front of the base and Beatbo's tummy will light up with multi-color LED lights while Beatbo goes into dance mode moving his hips and nodding his head. Three modes – Dance n Move, Learning & Games and Record a Phrase where Beatbo remixes it into a song. Has three jacks so you can use your own capability switches if needed.

*Size: 12½"L x 10"W x 13"H. Requires 4 AA Batteries. Weight: 3 lbs.*

### **Operation:**

1. Beatbo requires 4 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types of batteries together.
2. The battery compartment is located under Beatbo's base. Carefully remove the small screw from the cover using a Phillips head screwdriver, and lift the cover off. Observing proper (+) & (-) battery polarity, install 4 AA size batteries into the holder. Replace the cover and screw. Next turn the On/Off toggle switch to On.
3. Activate any one of the three large switches on the front of the base and Beatbo's tummy will light up with multi-color LED lights while Beatbo goes into dance mode moving his hips and nodding his head. Three modes – Dance n Move, Learning & Games and Record a Phrase where Beatbo remixes it into a song.
4. Beatbo three jacks so you can use your own capability switches if needed. To use an external switch, plug your switch into the desired jack on the back of Beatbo's base. Make sure your switch is plugged in all the way. There should be no gaps. Activating your switch will replicate the corresponding built in switch on the front of the base.

### **Troubleshooting:**

**Problem:** Beatbo does not work.

**Action #1:** Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

**Action #2:** Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

### **Care of the Unit:**

Beatbo can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not use abrasive cleaners, as they will scratch the surface of the unit.**

**Do not submerge the unit, as it will damage the electrical components.**