
Floppy Bunny #150

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Floppy Bunny

Floppy Bunny hops and wiggles its nose when you activate your capability switch. This switch adapted plush toy is great for teaching cause and effect. Accepts 1/8" plugs. Color may vary due to manufacturer's availability. Requires 2 AA Batteries. Weight: 3/4 lb.

Operation:

1. Floppy Bunny requires 2AA batteries (Not included) to operate. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well.
2. Turn Floppy Bunny upside down to reveal the battery compartment. Install 2 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure it.
4. Connect Floppy Bunny to any external capability switch (not included) via the jack located at the end of the Floppy Bunny's cord. Floppy Bunny is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure plug your switch all the way in. There should be no gaps.
5. Actuate your external capability switch and the Floppy Bunny will spring to life! It will hop and wiggle its nose.
6. Floppy Bunny will remain on while the external capability switch continues to be actuated. Once you release the external capability switch, there will be a brief delay, and then Floppy Bunny will

turn off.

Troubleshooting:

Problem: Floppy Bunny does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that your switch is plugged in all the way. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Note:

Small children should be supervised when playing with this toy, as there are some small parts, which could be a choking hazard.

Care of Unit:

- Do not inhibit Floppy Bunny's movement, as this may cause irreversible damage.
- Do not lift Floppy Bunny up by his tail, head or legs, or drop Floppy Bunny.
- If you will not be using Floppy Bunny for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components.