
**OVAL TEXTURE MULTI-SENSORY
CENTER/SWITCH #725 & #726**

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Feels great!

Our deluxe Multi-Sensory Centers are fun for all ages. Designed to provide auditory, tactile and visual stimulation, each center is equipped with ultra-bright lights, vibration and music. Internal controls allow for numerous combinations of different sensory stimulation (for example, music only, music and vibration, vibration and lights). The Multi-Sensory Center can also be used as a capability switch to activate other devices. Sturdy, durable design comes with suction cups and a non-skid base. Requires 2 AA Batteries. Weight: 1¼ lbs.

OPERATING INSTRUCTIONS:

1. Carefully turn the unit over to reveal the battery compartment. Using a small Phillips-head screwdriver, remove the cover and install two “AA” batteries, being careful to observe proper polarity.
2. To turn on or off the desired stimuli (1 – vibration; 2 – music; 3 - lights), push the small white rocker switches to the on or (open position). Open position means off. Please see *Figure 1*.

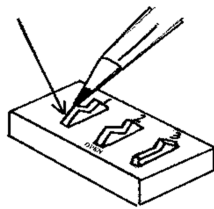


Figure 1

3. To use this unit as a switch to control other devices, plug the cord into the jack on the back of the unit and connect the other end of the cord into the device you wish to operate. If you leave the stimuli on, they will operate simultaneously with the external device.

IMPORTANT NOTES:

- If the lights, vibration or motor stop working, please replace the batteries (2 AA cells).
- Make sure there are no obstacles between the yellow plate and the housing unit.

If you have any problems or questions about this unit or any of our products, please call our **Technical Assistance Department**

Care of Unit:

The Multi-Sensory Center can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.