



Easy Flex Mount #1016

USER'S GUIDE

enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Increase independence. Our Easy Flex Mount has an adjustable Velcro strap which allows the user to access cups and bottles. It also has a flexible tube that can be positioned in a wide variety of angles and an easy-to-use clamp that attaches to wheelchairs, tabletops, strollers and high chairs. Cup holder sold separately. Size: 20"L. Weight: 1 lb.

Operation:

1. Attach the clamp to your mounting surface by first turning the clamp adjusting knob clockwise to loosen. Turn counter clock wise to tighten, do not over tighten or you will damage clamp. Once the Flex Arm is mounted you will want to adjust position of the arm for the best placement. Gently bend the arm to the desired position. To fine tune your placement of the item being held.
2. To attach an item to the Flex Arm such as a water bottle, open the Velcro strap and place your item. Secure the item with the Velcro strap as shown in the photos below and on back cover of this guide.

Please Note: The maximum weight the arm is designed to hold is between 1 to 2 lbs. Do not exceed this weight limit or damage to the arm and your item can occur.



Troubleshooting:

Problem: The clamp does not stay secure to the surface.

Action: Make sure that you have it clamped to an even surface so that the contact points have a flat surface to clamp to.

Problem: Flex arm droops and does not hold its position.

Action: Too heavy of an object (maximum weight 2 lbs.) is being attached to the arm and or the angle or bend in the arm is too extreme. Try a different position or use a lighter object.

Care of Unit:

The Easy Flex Mount can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a nontoxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.