

---

## Vibrating Plate Switch

---

### Gentle pulsations!

Our best-selling vibrating 5" x 8" plate switch is soothing to the touch and great for providing vibrotactile stimulation. Can be used alone or as an external switch when connected to a device. Requires 1 C Battery. Weight:  $\frac{3}{4}$  lb.

### Operation:

1. The Vibrating Plate Switch requires one C size battery. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The battery case is located under the unit. Install a new battery, being careful to ***observe battery polarity***.
3. This switch can be connected to any external toy via the 1/8-inch jack located on the back of the unit. The Vibrating Plate Switch #1040 comes with a 1/8- to 1/8-inch cord for activating external toys or devices with 1/8-inch jacks.

### Troubleshooting:

**Problem:** The vibrator does not activate or slows down.

**Action:** Replace the battery.

**Note:** This unit will still operate properly as a switch even if the batteries need replacing.

### Care of Unit:

The Vibrating Plate Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the electrical components.

**Do not use abrasive cleaners**, as they will scratch the surface of the unit.

---

---

## Vibrating Plate Switch #1040

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)