
Travelin' Turtle #1135

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

He's been everywhere!

Travelin' Turtle sports a camping hat and a shell fun of patches from his travels. Activate your switch and the animated turtle sings "I've Been Everywhere Man". Engages everyone with his fun animation and singing. Works with or without a switch. Size: 7"L x 7"W x 10"H. Requires 3 AA Batteries. Weight: 1 lb.

Operation:

1. Travelin' Turtle requires 3 AA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located on the bottom of the plush doll. Open the Velcro flap, then using a Phillips head screwdriver open and remove the batter cover. Next install 3 AA batteries paying close attention to proper battery polarity.
3. Travelin' Turtle to any external capability switch (not included) via the jack located at the end of his cord. Travelin' Turtle is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have pushed the switch plug all the way in. There should be no gaps.
4. Actuate your external capability switch or squeeze Travelin' Turtle's foot with the musical note symbol and he will sings "I've Been Everywhere Man". Engages everyone with his fun animation and singing. Great for teaching cause and effect and for developing visual attention.
5. Each switch activation or foot squeeze will start or stop Travelin' Turtle. If you hit the switch or squeeze his foot again during play, the rhymes and music will start from the beginning.

Troubleshooting:

Problem: Travelin' Turtle does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch is pushed in all the way. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Travelin' Turtle for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Travelin' Turtle is surface washable only.