

---

---

**Animal Massagers - #'s  
1142 Beetle, 1147 Turtle**

---

---

**USER'S GUIDE**



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

## Fun and stimulating!

Our spectacular Animal Massagers provide extraordinary somatosensory stimulation. Simply push your capability switch and they will massage your muscles and help you relax. Size: 5"L x 3"H. Requires Capability Switch and 2 AA Batteries. Weight: ½ lb.

### Operation:

1. Carefully remove the battery compartment cover on the massager's back. Install 2 "AA" sized batteries into the compartment paying close attention to proper polarity. Replace the battery compartment cover.
2. Plug any capability switch into the female jack on the massager. Any of our switches will work. If your switch has a 1/8" plug, it will fit directly into the jack. If your switch has a 1/4" plug, use one of our No.1160 low-profile adapter to connect to the massager. Make sure that all connections (between the switch and the adapter, between the adapter and the massager or between the switch and the massager) are very TIGHT. If there are any gaps, the massager may not operate correctly. Once you have everything connected, you can actuate the switch to activate the massager.
3. The massager will only run for as long as your switch is activated, once you release your switch the massager will stop running.

### Troubleshooting:

**Problem:** The unit fails to operate, or operates incorrectly.

**Action #1:** Check the batteries. Make sure they are still fresh. If you have a battery tester, test to make sure they are not dead or weak. Also check to make sure that they are installed according to proper polarity.

**Action #2:** Make sure that all connections are tight. There should be no gaps at all in the connections between the switch and the massager.

**Action #3:** Test your switch and/or your adapter (if applicable) with another device to rule out as the source of the problem.

### Care of the Unit:

The Animal Massager can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.

**Do not submerge** the unit, as it will damage the electrical components.