
Nursery Rhyme Caterpillar #1203

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Listen and learn with textures!

Come and learn your favorite nursery rhymes with our brightly colored textured friend. Activate each of the tactile switches and listen to the pre-recorded nursery rhymes recited by children while bright multi-colored lights twinkle along. Nursery rhymes include Mary Had a Little Lamb, Baa Baa Black Sheep, Humpty Dumpy, and Little Miss Muffet. Great for teaching cause and effect. Use as a tabletop toy or wall mount. Size: 23"L x 11"W x 4½"H. Requires 4 C Batteries. Weight: 2¼ lbs.

Operation:

1. Nursery Rhyme Caterpillar requires 4 C batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Turn the Caterpillar over, to reveal the battery compartment. Lift up on the Velcro attached cover to open the compartment and install 4 C batteries paying close attention to proper (+) & (-) battery polarity. Replace cover.
3. Turn the ON/OFF/VOLUME switch clockwise and set to desired volume level. When finished playing with Nursery Rhyme Caterpillar, turn the ON/OFF/VOLUME switch to OFF to help prevent battery drain. **Please Note:** If the ON/OFF/VOLUME switch is left ON, the batteries will drain out. For prolong periods of storage we recommend removing the batteries in the Caterpillar.
4. Each time you press one of the tactile switches, you will hear and learn one of four Nursery Rhymes while bright multi-colored LED lights twinkle along. When you press the Caterpillar face, the Caterpillar will ask you to come and learn your Nursery Rhymes with him.

Troubleshooting:

Problem: The Nursery Rhyme Caterpillar does not function correctly.

Action #1: Check and or replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure there is no debris or objects around the rim of the tactile switches that could cause them to jam or stick.

Care of Unit:

When finished using the Nursery Rhyme Caterpillar turn the ON/OFF/VOLUME switch to the OFF position.

If you will not be using the Nursery Rhyme Caterpillar for an extended amount of time, we recommend removing the batteries.

Nursery Rhyme Caterpillar can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.