

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure there is no dirt or obstacles between the rocking plate and the housing of the unit.

Action #2: Disconnect the power, by removing one of the batteries for 20 seconds, to reset the unit. Reconnect the battery. If the unit still does not work correctly, replace all batteries.

Care of Unit:

The Rocking Say It Play It can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

Rocking Say It Play It #1399

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Simultaneously activates toys and messages!

Great for developing correlation between language and function. Our Say It Play It plays two 10-second messages while simultaneously activating two toys or devices using the cords (included). Size: 14"L x 6"W x 2½"H. Requires 4 AA Batteries. Weight: 1¼ lbs.

Operation:

1. Carefully turn over unit and remove battery case cover to reveal battery compartment. Observing polarity, install four AA size batteries (Not included) into the holder. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. The microphone and the red "RECORD" button are located on the back panel of the unit. To record, depress and hold the red "RECORD" button. At the same time, depress one side of the rocking plate and speak into the microphone. Release both the plate and the recording button when the recording is over. Play back your message by pressing and releasing the same side of the plate used to record. Program the other side of the plate in the same manner.

The unit will retain recorded information indefinitely, even if the battery is disconnected. Previously stored information will be erased only when a new recording is performed.

3. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the back panel of the unit.
4. The Rocking Say It Play it can be connected to one or two toys or devices via the 1/8-inch jacks on the back panel of the unit. The Rocking Say It Play It #1399 comes with 1/8- to 1/8-inch connector cords for activating toys or devices with 1/8-inch jacks

Important Notes:

1. The memory of this unit is capable of holding up to a total of 20 seconds of speech information. This memory is divided into two equal segments of 10 seconds each. The first segment is assigned to the blue side of the plate, and the second segment, to the yellow side. If recording of a longer (over 10 seconds) message is desired, it can be performed using the blue side of the plate. The yellow side, however, will not be available for recording, in this case.

If message length exceeds available memory amount (e.g., 20 seconds for the blue side and 10 seconds for the yellow side), it will wrap around over itself at its beginning.

2. The Rocking Say It Play it will shut off automatically after playback, to preserve battery life. However, the device still drains a very small current in this "sleep" mode. Therefore, if the unit is not going to be used for a few days, it is recommended that the unit be turned off, to extend overall battery life. This can be done by turning the "ON/OFF/VOLUME" switch counterclockwise, until it "clicks" off.