

---

---

# Rocking Two Message Say It Play It #1410

---

---

## User's Guide



### For Technical Support:

Call our Technical Service Department,  
Monday through Friday, 9am to 5pm (EST).  
1-800-832-8697  
customer\_support@enablingdevices.com



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
www.enablingdevices.com

---

## Rocking Two Message Say It Play It

---

### *Plays your message and device!*

When switch plate is pressed, message is played and external toy or device is simultaneously activated. Records two messages (10 seconds per message). Device has two switch outputs to activate 2 devices. Two 2" x 2" frames are attached to the switch plate so you can insert your own object. Size: 5"L x 6"W x 1½"H. Requires 4 AA Batteries. Weight: ¾ lb.

#### **Operation:**

1. Carefully turn over unit and remove the small screw from the battery compartment. Slide the battery compartment cover off. Observing polarity, install four AA size batteries (Not included) into the holder. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well. Replace the battery compartment cover and replace the screw.
2. Turn the unit on by turning the ON/OFF/VOLUME knob located on the back panel of the unit. To record, simply press and "hold" the red "RECORD" button, then hold down one side of the plate. While continuing to hold the plate down, speak your message. Release both REC button and plate when you finish recording. Record the other side of the plate in the same manner.
3. To play back your message, press lightly on the side of your choice. It is not necessary to continue holding the plate down when listening to the message. The unit will continue to playback the entire message, even if pressure on the plate is released.
4. The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed.
5. The two, icon holders provide ample space for words, symbols or pictures. These visuals should be cut or designed to fit in the square inset area of each plate, and then covered with the clear overlays

provided. To do this, simply bend the clear plastic overlay and slip the sides into the slots.

6. Playback volume can be adjusted using the "ON/OFF/VOLUME" control knob located on the back panel of the unit.
7. To operate as a switch, as well as a communication device, connect the supplied double ended 1/8" to 1/8" cords to one or two toys or devices via the 1/8-inch jacks on the back panel of the unit. When you press either side of the plate, not only will it play your message, it will also activate the toy/device connected. The toy/device will activate for as long as the plate is held. Once the plate is released it will shut off.

#### **Important Notes:**

The memory of this unit is capable of holding up to a total of 20 seconds of speech information.

#### **Troubleshooting:**

**Problem:** The unit fails to operate, or operates incorrectly.

**Action #1:** Make sure there is no dirt or obstacles between the plate and the housing of the unit.

**Action #2:** Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries.

#### **Care of the Unit:**

The Rocking Two Message Say It Play It can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.

**Do not submerge** the unit, as it will damage the electrical components.