
Wheeee Mote Car #1439

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Easy control!

This one button remote control car beeps, flashes and spins. Activate and hold your capability switch and the car will go straight, press again and the car turns. Works both with and without a switch. Size: 6"L x 5¼"W x 4"H. Car requires 4 AA Batteries; Remote requires 1 9-V Battery. Weight: 1¼ lbs.

Operation:

1. To install the batteries for the controller, turn the controller over, remove the small Phillips Head screw from the battery cover then lift the cover off. Install one 9-Volt battery making note to follow proper (+) & (-) battery placement. Replace cover and screw. Alkaline batteries are recommended for best range of use and performance of this toy.
2. To install the batteries in the Wheeee Mote car, using a Philips Head screwdriver remove the screw on the bottom of the car, then lift cover off. Next install four AA batteries following proper (+) & (-) battery placement. As earlier mentioned follow battery recommendation. Replace battery cover and screw. Turn the car's ON/OFF switch located on the bottom of the car to ON.
3. This controller allows the user to move the car forward, and turning the car in a clockwise direction. Pressing on the control button or your own switch will drive the car in the direction the care is facing. Once you release pressure on the pad or your switch the car will stop. Pressing and holding the pad on the controller or your switch again, will make the car turn in a counterclockwise rotation. Release the pad or switch and the car will stop. Pressing and holding the pad or switch will start the car moving in the direction the car is facing.
4. When you are finished using the car turn the ON/OFF switch to the OFF position. The controller does not have an ON/OFF switch, when not in use make sure noting is pressing the pad, as this will drain the 9-Volt battery in the controller regardless if the car is turned ON. If the Wheeee Mote car is not going to be used for a long time, e.g. one month or longer, we recommend removing the

batteries from the car and controller during storage to prevent battery drain or leakage, which can cause damage to the car and controller.

Troubleshooting:

Problem: The Wheeee Mote Car is not running.

Action: Make sure batteries in the car and the controller are fresh and installed following proper (+) & (-) battery placement.

Problem: The Wheeee Mote Car is running intermittently.

Action: Replace batteries in the remote control and car. Weak batteries will severely limit the normal transmitting range of your remote control.

Problem: Vehicle does not react, or reacts poorly to operation by transmitter.

Cause: High-frequency interference occurred near to high-voltage lines, transformers and some types of building.

Action: Avoid these if possible, or choose another place to operate the vehicle.

Problem: Loss of control.

Cause: Another radio control vehicle using the same frequency is operated near your vehicle. **Please Note: do not use two Wheeee Mote cars near each other.** Out of sight or the control range of distance. (15 to 20 feet.)

Action: Move to another place where radio frequency interference from another device or vehicle is not present.

Care of Unit:

The Wheeee Mote Car can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit in any fluids, as it will damage the contents and the electrical components.