
Learning Lantern #1463

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Two switch activations!

We've adapted this fun, musical lantern with two switch activations. Press one switch for sing-along songs and tunes, press the other for sounds of the night. Lantern combines seven sing-along songs, fun phrases, nature sounds and activities with learning about numbers, counting, animals, day and night and more. Lantern works with or without a switch, has two volume levels and an on/off switch.

Size: 3"L x 5½"W x 6½"H. Requires 3 AA Batteries. Weight: 1 lb.

Operation:

1. Learning Lantern requires 3 AA Batteries to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. To install the batteries turn Learning Lantern over and locate the small screws on the battery cover door. Using a Phillips head screwdriver open the compartment and install 3 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and secure with the screws.
3. Set the High/Low/Off volume switch to the desired level. Next connect to any, one or two external capability switches (not included) via the jacks located at the back of Learning Lantern. Learning Lantern is equipped with two 1/8" female mono jacks for use with external capability switches that have a 1/8" mono plug. Make sure your connections are plugged in all the way. There should be no gaps.
4. Set the selector switch on the side of the Learning Lantern to music, 123 or night time. One switch jack will replicate pressing on the Learning Lantern's top button and the second switch jack will replicate spinning the wheel on the front of Learning Lantern. The Learning Lantern will also work without switches by activating the locations mentioned earlier. For more information on features

and functions please reference the original manufactures instructions

here: http://service.mattel.com/instruction_sheets/W2241pr-0920.pdf

Troubleshooting:

Problem: Learning Lantern does not function correctly.

Action #1: Make sure that the connection between the external capability switch or switches and the jacks are plugged in all the way. There should be NO gaps.

Action #2: Try your switch with another switch operated device in an effort to rule out your switch as the source of the problem.

Action #3: Replace the batteries paying close attention to proper battery polarity.

Care of Unit:

If you will not be using Learning Lantern for an extended amount of time, we recommend removing the batteries.

Learning Lantern can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.