
Learning Lamp #1464

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Pretty in pink!

We've adapted this interactive lamp so when you activate your capability switch more than a dozen learning songs, tunes and phrases play. Learn counting, shapes, colors and more. Has volume control and two modes of play – learning and music. All buttons also work without a switch. Size: 8"L x 4½"W x 10"H. Requires 3 AA Batteries. Weight: 1¼ lbs.

Operation:

1. Learning Lamp requires 3 AA Batteries to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. To install the batteries turn Learning Lamp over and locate the small screws on the battery cover door. Using a Phillips head screwdriver open the compartment and install 3 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and secure with the screws.
3. Set the High/Low Volume switch to the desired level. Next connect to any external capability switch (not included) via the cord- jack. Learning Lamp is equipped with a 1/8" female mono jack for use with external capability switches that have a 1/8" mono plug. Make sure your connections are plugged in all the way. There should be no gaps.
4. Set the selector switch on the side of the Learning Lamp to ABC or Music. When you activate your capability switch more than a dozen learning songs, tunes and phrases play. Learn counting, shapes, colors and more. The Learning Lamp will also work without switches by activating the locations on the Learning Lamp itself. For more information on features and functions please reference the original manufactures instructions

Troubleshooting:

Problem: Learning Lamp does not function correctly.

Action #1: Make sure that the connection between the external capability switch and the jack is plugged in all the way. There should be NO gaps.

Action #2: Try your switch with another switch operated device in an effort to rule out your switch as the source of the problem.

Action #3: Replace the batteries paying close attention to proper battery polarity.

Care of Unit:

If you will not be using Learning Lamp for an extended amount of time, we recommend removing the batteries.

Learning Lamp can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non-toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.