

5. Once your mounting plate has been attached to the arm you can use the included Velcro to attach your switch, communicator or other device to the mounting plate.

Please Note: Maximum weight for this mounting system is 3Lbs.

Caution:

Do not over tighten the adjusting knobs, or try to move or adjust the mounting system after you have tightened the adjusting knobs. Doing so can damage the Mounting System and void your warranty.

Care of Unit:

The Mounting System can be wiped clean with any mild household multi-purpose cleaner and disinfectant. **Do not use abrasive cleaners**, as they will scratch the surface of the unit.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com

New Light Duty Magic Arm #1557

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Sleek & modern design!

Our Light Duty Magic Arm System has a single articulated arm with two sections and a heavy duty clamp. The arm is highly adjustable and securely mounts onto any wheelchair, tabletop or bedrail. Holds up to 3.3 lbs. Arm expands from 14"L to 24"L for maximum flexibility and reach. Two mounting plates available separately. Weight: 2¼ lbs.

1. **Mounting Arm Assembly:** Your mounting system consists of two main parts, an adjustable arm and a clamp. First you will need to assemble the 2 main parts. To attach the clamp to the arm, loosen the set screw on the clamp (see Fig. 1). Next depress the Silver Spring loaded locking pin, then push one of the ends of the mounting arm (with the 3/8" threaded end into the clamp) (see Fig. 2) Please Note: you need to line up the flat part of the Mounting Arm end post with the set screw in the clamp. This will prevent the arm from spinning freely inside the clamp. Once this is done you can tighten the setscrew to lock the Mounting arm to the clamp.



Fig. 1

2. **Clamping Options:** Our mounting clamp is designed for mounting to tubular (wheelchair, bedrail, walker, etc.) or flat surfaces such as a table top or desk. For flat surface mounting use the included wedge insert shoe stored in front of the locking knob on the clamp (see Fig.2) Loosen locking knob slightly to remove wedge, tighten knob once wedge has been removed.



Fig.2

3. You can now place the desired mounting plate (Not included #1569 Set) on the end of the mounting arm. Simply screw the plate into the threaded opening at the top of the mounting arm until tight. This can be done by hand. The mounting arm has two different size threaded ends, one is a 3/8" the other is a 1/4". Our mounting plates use the 1/4" size. (See Fig. 3).

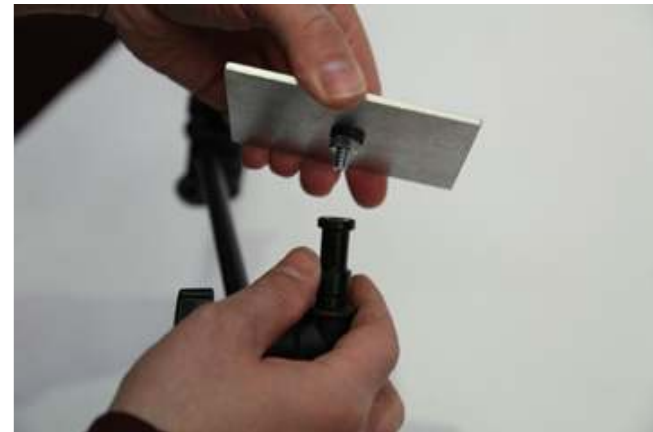


Fig.3

4. Do not force or try adjusting the arm without first loosening the set knobs for that portion of the arm. Each section of the arm has a set knob. Once you have the arm in the desired position you can tighten the knobs to lock the arm in place.