
Meowsic #1662

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Make music!

We've adapted the two song buttons on this delightful keyboard and added a sturdy removable base for accessibility. Press your capability switch to play 20 familiar songs, press your second capability switch to play 7 witty kitty songs. Device also allows you to play all the songs with 5 different instrument buttons and 5 different rhythm sounds. Comes with sing along microphone and volume and tempo control as well as recording capability. Great for creating music, music appreciation and improving listening skills. Size: 19"L x 10"W x 2³/₄"H. Requires 4 AA Batteries. Weight: 2¹/₂ lbs.

Operation:

1. Meowsic requires 4 AA batteries (Not included). Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. To install the batteries turn Meowsic over and locate the small screw on the battery cover door. Using a Phillips head screwdriver open the compartment and install 4 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure with the screw. There are two external switch input jacks for plugging in your own switches (not included) on the back of the unit. Meowsic is equipped with two 1/8" female mono jacks for use with external capability switches that have a 1/8" mono plug. Make sure your switches are plugged in all the way. There should be no gaps.
4. Please reference the included original manufactures instructions for all of Meowsic functions and features.

5. Press one of your capability switch to play 20 familiar songs, press your second capability switch to play 7 witty kitty songs. Device also allows you to play all the songs with 5 different instrument buttons and 5 different rhythm sounds. Comes with sing along microphone and volume and tempo control as well as recording capability. Great for creating music, music appreciation and improving listening skill
6. Meowsic also has a Velcro attached removable base allowing you to place Meowsic flat on a table top or wheelchair tray.

Troubleshooting:

Problem: Meowsic does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack are plugged in all the way. There should be NO gaps.

Action #3: Try your switches with another switch operated toy/device in an effort to rule out your switch as the source of the problem.

Care of Unit:

If you will not be using Meowsic for an extended amount of time, we recommend removing the batteries.

Meowsic can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.