

---

---

## Happy Pin Switch Vibration & Music #209

---

---

### USER'S GUIDE



#### For Technical Support:

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)


## Happy Pin Switch w/Lights, Vibration & Music

### Build self-esteem with lively stimulation!

The slightest touch rewards the user with bright lights, happy music and soothing vibration. The individual rewards can be turned on or off by a hidden switch. The unit will act as a switch for 4 devices (four cords are included). Size: 7½"L x 8¼"W x 9"H. Requires 2 AA Batteries. Weight: 1½ lbs.

### OPERATING INSTRUCTIONS:

1. Carefully turn the unit over to reveal the battery compartment. Using a small Phillips-head screwdriver, remove the cover and install two "AA" batteries, being careful to observe proper polarity.
2. To turn on or off the desired stimuli (1 – vibration; 2 – music; 3 – lights), push the small white rocker switches to the on or (open position). Open position means off. Please see Figure 1.
3. Moving the Happy Pin in one of four directions using a clock face as an example (12 o'clock, 3 o'clock, 6 o'clock and 9 o'clock), will activate the stimuli that is selected as well as a device for each of the corresponding jacks located on the back of the unit.

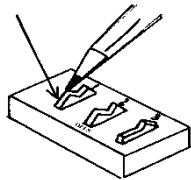

*Figure 1*

4. To use this unit as a switch to control other devices, plug one of the four supplied 1/8" to 1/8" cords into one of the jacks on the back of the unit and connect the other end of the cord

into the device you wish to operate. If you leave the stimuli on, they will operate simultaneously with the external device.

### IMPORTANT NOTES:

- If the lights, vibration or motor stop working, please replace the batteries (2 AA cells).
- Make sure there are no obstacles between the Happy Pin's base plate and the housing unit.
- When connecting a toy/device, be sure there are no gaps between any of the connections. All should be flush.
- Try your toy/device with a different switch to rule out the Happy Pin Switch as the source of the problem.

If you have any problems or questions about this unit or any of our products, please call our **Technical Assistance Department** at:

**(1-800-832-8697)**