

Mounting Options:

The Peek-A-Boo Wall Panel has 4 mounting holes for wall mounting.

Please Note: Mounting hardware is not included. You can use standard wall anchoring hardware found at hardware stores and Home Improvement Centers. Please consult one of their hardware associates for the proper mounting hardware based on your wall construction. In addition we also offer our Track System when mounting 3 or more Panels item No. 3116.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

Peek-A-Boo Wall Panel #2265

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

As you move the bead chains to look in the mirror, red lights will blink while music box music plays. On/off vibration. Great for sensory exploration. *Size: 13"L x 15"W x 3¾"H. Requires 4 C Batteries. Weight: 4 lbs.*

1. The Peek-A-Boo Wall Panel requires 4 C batteries (not included). Use Alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. The battery compartments are located under the magnetically held compartment on the top of the panel. To access the compartments lift up the access door and then remove the Phillips Head screws from each battery compartment cover. Install the C batteries into each holder. When installing new batteries, always be careful to observe battery polarity. Replace battery covers and screws. Never mix old and new batteries together or different brands or types together. (See Photo).



2. Turn the ON/OFF/VOLUME knob clockwise, you will hear an audible click. Adjust to knob for desired volume level of the music. There is also an ON/OFF toggle Vibration switch for controlling the vibration. This will add vibration when you move your hands across the beads. The Peek-A-Boo Wall Panel can be used with or without the vibration function. **Please Note:** Each time you move the bead chains to look in the mirror, red lights will blink while music box music plays.

Please Note: Some low room light required for activation.

Troubleshooting:

Problem: The Peek-A-Boo Wall Panel's lights, music or vibration do not work when activated.

Action #1: Make sure the ON/OFF/VOLUME knob and the Vibration ON/OFF toggle switch is turned.

Action #2: Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: The Peek-A-Boo Wall Panel does not play music.

Action #1: Make sure the ON/OFF/VOLUME knob is turned ON and set at an audible level.

Action #2: Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: The Peek-A-Boo Wall Panel does not work at all.

Action: Check all Batteries for strength and for proper battery polarity. Replace if weak or dead.

Care of the Unit:

The Peek-A-Boo Wall Panel can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.