
Bubble Mania #2286

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Flood your world with bubbles!

We found a bubble machine which will provide hours of enjoyment. Activate the machine with your switch and love the streams of bubbles. A real fun way to increase visual tracking skills. AC line cord has been removed for safety reasons. Size: 9"L x 7"W x 5½"H. Requires Capability Switch and 2 C Batteries. Weight: 3¼ lbs.

Operating instructions:

1. Bubble Mania requires 2 C batteries (Not included) to operate. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well.
2. The battery compartment is located on the top of the unit. Depress the latch on the over and remove. Next install 2 C batteries paying close attention to proper (+) and (-) battery polarity.
3. Replace the battery compartment cover. Connect Bubble Mania to any external capability switch (not included) via the jack located at the back of the unit. Bubble Mania is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have a tight connection. There should be no gaps.
4. You are now ready to fill your Bubble Mania with bubble juice. **Please Note: Bubble juice is not included. You can use any good quality bubble liquid, or purchase our gallon of bubble juice catalog No.2282.** Fill the bubble tray to approximately ½" from the top.
5. Bubble Mania can be used with or without an external switch. The ON/OFF switch is located in the back of the unit. When using an external capability switch, the ON/OFF switch needs to be set in the OFF position. Actuate your external capability switch and the bubbles will start to flow! Once you release your switch the bubbles will stop.

Please Note: Please Note: AC adapter has been removed for safety reasons. This unit will work on battery power only.

Troubleshooting:

Problem # 1: The Bubble Mania does not turn on when switch is depressed.

Action # 1: Make sure that you press all connections firmly together. Any gaps may interrupt electrical connection.

Action # 2: Make sure batteries are fresh.

Action # 3: Try a different switch with your Bubble Mania to rule it out as the source of the problem.

Problem #2: Bubble Mania does not blow bubbles.

Action: Make sure you have bubble juice in the Tray.

Care of Unit:

The Bubble Mania can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.