



enabling
devices

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com

Wrist Talker 3037 & 3037B

USER'S GUIDE

enabling
devices

50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Wrist Talker

Easy to wear!

This single message communicator is easy to use.

- Record message of up to 10 seconds
- Great sound quality
- Enables the user to communicate any greeting
- Easy to record and re-record
- Has an adjustable strap

Size: 2¼"D x 1½"H. Requires 3 LR44

Button Cells (Included). Weight: ¼ lb.

Operation:

1. Wrist Talker requires 3 LR44 Button cell batteries to operate (Included). **Please Note: Before use remove and discard the round foam disk with hole in the center. This was used to prevent battery draining during shipping.**

To Record:

2. To make a recording simply twist the cover until the opening lines up with the gray rubber record button (see illustrations). Next depress the record button and speak your message from six to eight inches away from it. When finished recording release the record button. You have a maximum record time of ten seconds.

To Replace Batteries:

3. Turn the Wrist Talker over, and then carefully remove the black Velcro wrist strap (see illustrations). Next with one hand turn the unit back over, catch the white disk and voice module in your other hand as it drops out. There are three tabs on the voice module that need to be depressed in order to replace the batteries. Hold the voice module with the speaker side facing down. Next with your thumb nail press in and lift gently on the three tabs on the sides of the cove one at a time.
4. **Please Note:** On the body of the voice module you will see three grooves, just above them are the tabs you need to depress (see illustrations). Once this is done the cover should lift off. Observing proper (+) and (-) battery polarity, install three LR44 button cell

batteries into the holders (+ side facing up). Replace the speaker and circuit board, making note to line up the grove in the speaker and circuit board with the grove in the white casing (see illustrations). If this is not done the unit will not go back together properly.

Message Playback:

4. To play back your message slid the cover plate back so that the microphone is showing through the opening in the cover. Next depress the center white button.
5. When placing the recording modal back into the wrist holder face the speaker side towards you. Next place the round disc in place and feed the wrist strap through both slots. When the wrist talker is on your wrist the speaker should be facing down towards your wrist and the message play-back button should be facing up on top.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure that you have assembled all the parts correctly according to the instructions and illustration sheet.

Action #2: Check your batteries – There is a 99% chance that either one is installed incorrectly and/or one is weak/dead.

Action #3: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries paying careful attention to proper polarity.

Care of the Unit:

The Wrist Talker can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.

Rev 4/27/17