
Soft Shaggy Switch #3133

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Easy activation!

Our Soft Shaggy Switch activates devices with the slightest touch. The long pile shaggy fur is so soft it makes everyone want to reach out and touch it. Fabric is removable for washing. Size: 7"L x 5"W x 2"H. Weight: ¼ lb.

Operation:

1. Connect the Soft Shaggy Switch to any switch adapted device via the 1/8" cord plug on the Soft Shaggy Switch. Make sure you have pushed the switch plug all the way in. There should be no gaps.
2. Applying slight pressure to the Soft Shaggy Switch will cause activation of your device. Activation will only be maintained under sustained pressure of the Soft Shaggy Switch.
3. The internal switch is removable from the Soft Shaggy cover for cleaning of the cover. Locate the Velcro seam where the switch cord exits the Soft Shaggy Switch. Open the seam and remove the switch. See care of unit section for washing instructions.

Troubleshooting:

Problem: The Soft Shaggy Switch does not activate a connected device.

Action #1: Make sure all connections are plugged in all the way.

Action #2: Check batteries in your device, replace if weak or dead.

Care of Unit:

You can hand wash the Shaggy Switch cover with a mild laundry detergent or machine wash on a delicate cycle setting. Air dry after washing.

Do not submerge the switch unit in any fluids, as it will damage the contents and the electrical components.