
Cushion Grip Switch #3137

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Sizes!

A fantastic sensitive switch that fits most hands. Just slip it over your hand and you can use a palmer grasp or roll it on a tray or tabletop to activate any device or toy. Size: 5"L x 2"W x 2"H. Weight: ¼ lb.

Operation:

1. Connect the Cushion Grip Switch to any switch adapted device via the 1/8"cord plug on the Cushion Grip Switch. Make sure you have pushed the switch plug all the way in. There should be no gaps.
2. Slip the Cushion Grip Switch onto one of your hands you can use a palmer grasp or roll it on a tray or tabletop to activate any switch adapted device or toy.
3. Applying slight pressure to the Cushion Grip Switch will cause activation of your device. Activation will only be maintained under sustained pressure of the Cushion Grip switch.

Troubleshooting:

Problem: The Cushion Grip Switch does not activate a connected device.

Action #1: Make sure all connections are tight.

Action #2: Check batteries in your device, replace if weak or dead.

Care of Unit:

The Cushion Grip Switch can be wiped clean with a damp cloth if needed.

Do not submerge the unit in any fluids, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.