
Sound Pillow #3140

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Sound and vibration!

We've switch adapted the sound machine in this vibrating pillow. Sound machine has four sound choices: music, water, white noise and a heartbeat. Simply remove the sound machine from the interior of the pillow, select a sound choice and attach your switch. Replace the machine and now the user can turn the sound on and off. Pillow has two-speed vibration and is covered with a soft, comfortable corduroy fabric. Sound machine has a 15 minute auto timer shut off. Pillow size: 14"L x 9½"W x 3½"H. Pillow requires 2 D Batteries; Sound machine requires 3 AAA Batteries. Weight: 1 lb.

Operation:

1. To install the two D batteries and the 3 AAA batteries in the Sound Machine, you must locate the zipper on the back of the Pillow. Next unzip the opening and separate the opening in the foam pillow to reveal the battery holder. Install the two D batteries paying close attention to proper (+) and (-) polarity.
Please Note: Alkaline batteries are recommended (e.g. Duracell or Energizer brand) Do not use rechargeable or heavy duty batteries as they will not work as well with this device. Never mix old and new batteries together.
2. To install the Sound Machine batteries use a Dime, or flathead screwdriver to open the battery compartment and install 3 AAA batteries.
3. Vibration Speed control switch has three positions, Low, Off and High. The center position, this is the OFF position.
4. Connect the Sound Pillow to any external capability switch (not included) via the cord jack located at the back side of the pillow. Sound Pillow is equipped with a 1/8" female mono jack for use with external capability switches that have a 1/8" mono plug. Make sure you plug your switch in all the way. There should be no gaps.

5. The Sound machine has four sound choices: music, water, white noise and a heartbeat. Simply remove the sound machine from the interior of the pillow, select a sound choice and attach your switch. Replace the machine and now the user can turn the sound on and off. Actuate your capability switch to turn the selected sound on and activate it a second time to turn it off. Sound machine has a 15 minute auto timer shut off.

Troubleshooting:

Problem: The Sound/Vibrating Pillow does not activate.

Action #1: Make sure that all batteries are installed following proper (+) and (-) polarity.

Action #2: Replace the batteries with new alkaline batteries (E.G Duracell or Energizer Brand).

Action #3: Do not use rechargeable or heavy duty batteries.

Care of Unit:

The Sound Pillow is surface washable only.

Do not submerge the unit, as it will damage the contents and the electrical components.