

Mounting Options:

The Ocean Gel Wall Panel has 4 mounting holes for wall mounting. **Please Note:** Mounting hardware is not included. You can use standard wall anchoring hardware found at hardware stores and Home Improvement Centers. Please consult one of their hardware associates for the proper mounting hardware based on your wall construction.



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com

Kaleidoscope Wall Panel #3280

USER'S GUIDE



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Dazzling display!

See a dazzling display of rotating multicolored lights when the panel is activated. Just move your hand in front of the motion sensor panel to light it up and play soft, soothing music. Music plays for 20 seconds after activation. Encourages exploration and directed reach. A great addition to any sensory space. Wall mounting hardware included. Size: 18"L x 16"W x 4½"H. Requires 6 C Batteries. Weight: 4¼ lbs.

1. The Kaleidoscope Wall Panel requires 6 C batteries (not included). Use Alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. The battery compartments are located under the compartment on the top of the panel. To access the compartments lift up the access door and then remove the Phillips Head screws from each battery compartment cover. Install the C batteries into each holder. When installing new batteries, always be careful to observe battery polarity. Replace battery covers and screws. Never mix old and new batteries together or different brands or types together. (See Photo).



2. Turn the ON/OFF/VOLUME knob clockwise, you will hear an audible click. Adjust to knob for desired volume level of the music.

3. See a dazzling display of rotating multicolored lights when the panel is activated. Just move your hand in front of the motion sensor panel to light it up and play soft, soothing music. Music plays for 20 seconds after activation. Encourages exploration and directed reach.

Please Note: Some low room light required for activation.

Troubleshooting:

Problem: The Kaleidoscope Wall Panel's lights or music do not work when activated.

Action #1: Make sure the ON/OFF/VOLUME knob is turned ON and adjusted to an audible level.

Action #2: Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

Problem: The Kaleidoscope Wall Panel does not work at all.

Action: Check all Batteries for strength and for proper battery polarity. Replace if weak or dead.

Care of the Unit:

The Kaleidoscope Wall Panel can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.