
Drop-in-a-Bucket #349

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Easy to reach!

This containerizing toy has been designed with a low profile so you only have to slightly reach to drop objects into the bucket. Six multicolored lights draw the user's attention. When they place an object in, a tune plays. Comes with interchangeable grids to teach object placement. Terrific for encouraging size and shape recognition. Size: 12"L x 5½"W x 7"H. Requires 4 AA Batteries. Weight: 1¾ lbs.

Operation:

1. Drop-in-a- Bucket requires 4 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Gently turn the unit over to face the black battery compartments. Carefully remove the small screws from the battery compartment covers using a Phillips head screwdriver, and slide the covers off. Observing proper (+) & (-) battery polarity, install four AA size batteries into the holders. Replace the battery compartment covers and screws.
3. Turn the ON/OFF switch to the ON position. This will turn ON the multi colored LED lights on the top of the bucket. Next place one of the included shape grids on top of the bucket.
4. Each time the user places a shape into the appropriate opening, they will be rewarded with a brief sampling of music, this will encourage the user to place more shapes to continue hearing the music.

Please Note:

When finished playing, turn the ON/OFF switch to OFF to prevent the lights from draining down the batteries.

Troubleshooting:

Problem: There are no lights or music.

Action #1: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

Action #2: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Care of the Unit:

Drop-in-a-Bucket can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.