



Mini Mount w/Mini Gumball Switch #3709

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Mounts anywhere!

This sturdy clamp and gooseneck has an easily adjustable ball joint with a Mini-Gumball switch. Mounts to tabletops, wheelchair trays or bedrails. Size: 14"H. Weight: 1 lb.

Mounting Setup:

1. The base clamp comes with two removable rubber clamping pads. One is set in place and the second is stored on the back of the clamp. Choose the pad that gives the clamp the best hold for your mounting application. (See photos). Next attach the clamp to your mounting surface.



2. Once the Gooseneck is mounted you will want to adjust position or the angle of the gooseneck for the best placement. Gently bend the Gooseneck to the desired position. To fine tune your placement of the switch, loosen the set knob on the ball-joint. Next adjust the switch to the desired position and then tighten the set knob. (See Photo).



Switch Operation:

1. Plug the Mini Gum Ball Switch into the jack on your switch-adapted toy/device.
2. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
3. Make sure that the connection between the Mini Gum Ball Switch and your toy/device is tight. There should be no gaps.
4. Simply press anywhere on the Mini Gum Ball Switch's colored top to activate your toy/device.
5. The toy/device will only remain activated while the Mini Gum Ball Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Mini Gum Ball Switch does not activate your toy/device.

Action #1: Make sure that the connection between the Mini Gum Ball Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #3: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Mini Mount can be wiped clean with any mild household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Rev 5/4/17