
Chihuahua #3928

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

A cute pet!

Take your new companion out for a walk. He walks forwards, backwards, barks and wags his tail. He is extremely lively and adventurous. He loves to get tons of affection too. Size: 6"L x 5"W x 7"H. Requires 2 AA Batteries. Weight: ½ lb.

Operation:

1. Chihuahua requires 2 AA Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
2. Turn Chihuahua upside down to reveal the battery compartment. Remove the small screw with a Phillips Head screwdriver. Install 2 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and screw.
3. Connect Chihuahua to any external capability switch (not included) via the jack located at the end of Chihuahua's cord. Chihuahua is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to make a tight connection. There should be no gaps.
4. Actuate your external capability switch and Chihuahua will spring to life! He walks forward and backwards, barks and wags his tail.
5. Chihuahua will remain on while the external capability switch continues to be actuated. Once you release the external capability switch Chihuahua will turn off.

Troubleshooting:

Problem: Chihuahua does not function correctly.

Action #1: Replace the batteries paying close attention to proper (+) & (-) battery polarity.

Action #2: Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

- Do not inhibit Chihuahua's movement, as this may cause irreversible damage.
- Do not lift Chihuahua up by his head, tail or legs, or drop Chihuahua.
- If you will not be using Chihuahua for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Chihuahua is surface washable only.