# **Baby Pug #3929**

# **USER'S GUIDE**



# For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

#### Take him home!

Press and hold your capability switch and this adorable Baby Pug walks forward, scoots backwards and barks. Small, cute and soft. Size:  $3\text{W} \times 6\text{L} \times 5\frac{1}{2}\text{H}$ . Requires 2 AA Batteries. Weight:  $\frac{1}{4}$  lb.

## Operation:

- 1. Baby Pug requires 2 AA Batteries to operate. Alkaline batteries must be used (e.g. Duracell or Energizer Brands). Do not use rechargeable batteries because they supply a lower voltage and the unit may not perform as well. Never mix old and new batteries together or different brands or types together.
- 2. Turn Baby Pug upside down to reveal the battery compartment. Remove the small screw with a Phillips Head screwdriver. Install 2 AA batteries paying close attention to proper battery polarity. Replace the battery compartment cover and screw.
- 3. Connect Baby Pug to any external capability switch (not included) via the jack located at the end of Baby Pug's cord. Baby Pug is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure to plug your switch all the way in. There should be no gaps.
- 4. Actuate your external capability switch and Baby Pug will spring to life! He walks forward and backwards, barks and wags his tail.
- 5. Baby Pug will remain on while the external capability switch continues to be actuated. Once you release the external capability switch Baby Pug will turn off.

### **Troubleshooting:**

**Problem:** Baby Pug does not function correctly.

**Action #1:** Replace the batteries paying close attention to proper (+) & (-) battery polarity.

**Action #2:** Make sure that the connection between the external capability switch and the jack is TIGHT. There should be NO gaps.

**Action #3:** Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

#### Care of Unit:

- Do not inhibit Baby Pug's movement, as this may cause irreversible damage.
- Do not lift Baby Pug up by his head, tail or legs, or drop Baby Pug.
- If you will not be using Baby Pug for an extended amount of time, we recommend removing the batteries.

**Do not submerge** the unit in water, as it will damage the electrical components. Baby Pug is surface washable only.