Hey Diddle Diddle #4902

USER'S GUIDE



For Technical Support:

Call our Technical Service Department Monday through Friday, 9 a.m. to 5 p.m. (EST) 1-800-832-8697

customer support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Cat and fiddle!

Features fun sound effects, a blinking light, and the familiar "Hey Diddle Diddle" melody. We added four brightly colored accessible switches to control all of the activities. Comes on a sturdy base. Great for teaching cause and effect. Size: $10\frac{1}{2}$ "L x 3"W x $9\frac{1}{2}$ "H. Requires 2 AA Batteries. Weight: 1 lb.

Operation:

- Hey Diddle Diddle requires 2 AA batteries. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types of batteries together.
- 2. The battery compartment is located on the back of the Hey Diddle Diddle. Carefully remove the small screw from the cover using a Phillips head screwdriver, and lift the cover off. Observing proper (+) & (-) battery polarity, install 2 AA size batteries into the holder. Replace the cover and screw.
- 3. Next set the ON/OFF/Sound Effects/ Music switch to the desired mode. Press any one of the colored buttons or the characters directly to hear a different sound or the Hey Diddle Diddle nursery rhyme. Each switch the music mode features a different character at the end of the nursery rhyme. When finished using turn the ON/OFF switch to OFF to prevent battery drain.

Troubleshooting:

Problem: Hey Diddle Diddle does not work.

Action #1: Rotate (spin) the batteries within the battery compartment to ensure a good connection. Check that all batteries are installed properly.

Action #2: Check all batteries for freshness, if weak or dead replace with new batteries. Use an alkaline battery (E.G. Duracell or Energizer brand).

Care of the Unit:

Hey Diddle Diddle can be wiped clean with any mild household multipurpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.