
Wearable Talker #5022

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Two messages!

Our Wearable Talker is so versatile; you can wear it on any part of your body—your wrist, forearm, or leg – or place it on a wheelchair arm.

Features:

- 20 seconds of recording time
- 10 seconds per message
- 2 messages
- Adjustable volume
- Two clear covers for switches
- Extra add-on strap

Size: 5½"L x 2¼"W x 1½"H. Requires 3 AAA Batteries. Weight: ¼ lb.

Operation:

1. The Wearable Talker requires 3 AAA batteries to operate (Not included). There are two Phillips head screws one on each side of the talker. Remove each screw to open the unit and reveal the battery compartment. Lift the bottom cover off with the help of the wrist strap. Observing polarity, install 3 AAA size batteries into the holder. Alkaline batteries must be used (e.g. Duracell or Energizer brand). Do not use rechargeable or heavy duty batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Close the unit back up and replace screws.
2. The Wearable Talker can record two messages up to 10 seconds for each. There is a separate Microphone and Record button for each message button. To record your message, turn the Talker's ON/Off/Volume knob about midway. Next press and hold the Record button for the corresponding message button then while holding the record button press the message button and speak your message into the Microphone. Speak between 6 to 8 inches away from the Microphone for optimal recording. Repeat step for second message button.
3. To play back your message, simply press the button you just recorded. It is not necessary to continue holding the button when listening to the message.
4. The unit will retain recorded information indefinitely, even if the batteries are disconnected. Previously stored information will be erased only when a new recording is performed.

5. Talker advances each time the two colored buttons are pressed. When you come back around like a continuous loop you will hear the changed message.

The Wearable Talker comes with an extra extension strap so you are able to use it on your upper arms, legs etc.

Please Note: If the Wearable Talker is not going to be used for a long period of time, e.g. one month or longer, we recommend removing the batteries during storage to prevent battery drain or leakage, which can cause damage to the unit.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Disconnect the power by removing one of the batteries for 20 seconds in order to reset the unit. Re-install the batteries. If the unit does not work correctly, replace all the batteries paying careful attention to proper polarity.

Action #2: Make sure that you are holding both the record switch and the message button simultaneously while you are recording.

Action #4: Make sure that there is nothing covering the Microphone.

Problem: The unit is playing old messages fine, but will not record new messages.

Action #1: Check your batteries – There is a 99% chance that batteries are weak or are non-Alkaline.

Care of the Unit:

The Wearable Talker can be wiped clean with any mild household multi-purpose, non-abrasive cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Do not submerge the unit, as it will damage the electrical components.