
Visually and Hearing Impaired Activity Center

Provides hours of somatosensory stimulation!

This activity center is specially-designed to introduce individuals who cannot see or hear, to the world of play. Includes a spinning soft cloth, a vibrating plate and a fan. Size: 13"L x 9"W x 4½"H. Requires 2 C Batteries. Weight: 1½ lbs.

Operation:

1. The Visually and Hearing Impaired Activity Center requires two "C" size batteries. The battery compartment is located under the unit. Install new batteries, being careful to observe proper battery polarity. The batteries will fit snugly. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well.
2. When you press on the blue plate, you will be rewarded with soothing vibration. Press anywhere on the blue plate **BELOW** the level of the two screws.
3. Gently tug on the wooden pull ball and air will blow over your hand from the built in fan.
4. Actuate the textured yellow oval plate switch to spin the soft cloth. (Exercise caution with long loose hair - We recommend tying it back in a ponytail)
5. All three of these functions will only remain "activated" while you maintain pressure on the corresponding switch. Once you release pressure, the feature will turn off.

Troubleshooting:

Problem: The unit stops functioning, or functions incorrectly.

Action: Replace batteries paying close attention to proper battery polarity.

Problem: One of the features is not functioning or is running constantly.

Action: Make sure that there is nothing obstructing/inhibiting the function of the corresponding switch (either holding it down or preventing it from being actuated).

Care of Unit:

The Visually and Hearing Impaired Activity Center can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Visually and Hearing Impaired Activity Center #540

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com