

---

## Multisensory Motivational Center #555

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

[www.enablingdevices.com](http://www.enablingdevices.com)

## Therapeutic!

This complete cause and effect motivational center has been specially designed for use in therapeutic and educational environments. The center includes three different toys that are activated by interchangeable switch pads. The child has a choice of using a round, square, or triangle plate. The pads give visual and tactile cues for the child, while the toys atop the center provide auditory, visual, and somatosensory stimulation. The package comes with three battery operated toys, switch pads and three cords so that any other adapted toy can be used. Size: 22"L x 11"W x 2"H. Requires 4 AA Batteries and 2 C Batteries. Weight: 5 lbs.

The toys, which attach with Velcro to the top of the unit, are activated by light pressure on three differently colored, Velcro-covered, built-in switches. The three switch pads can be interchanged with an oval, square and triangular plate (attachable with Velcro).

This center provides an environment for developing fine and gross motor planning, as well as eye-hand coordination. The switch pads give visual and tactile cues for a child, while the toys on top of the center provide auditory, visual, and somatosensory stimulation.

Mounted on a sturdy base, this unit can be secured to any smooth surface by means of the attached suction cups.

## Operation:

1. The Multisensory Motivational Center does not require batteries; however, the toys that the unit activates do require batteries. Check each toy for the size of batteries required. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the toys may not perform as well.

2. To set up the unit connect a cord to each toy via the three jacks on the back of the unit. The Multisensory Motivational Center comes with 1/8-inch jacks, 1/8- to 1/8-inch connector cords, and toys with 1/8-inch jacks.
3. The Multisensory Motivational Center can also be used to activate any of our other specially adapted toys available. If you use 1/4- to 1/8-inch adaptors, they must be mono adaptors, not stereo.

## Troubleshooting:

**Problem:** The unit stops activating a toy.

**Action:** Replace the batteries in the toy.

## Care of Unit:

The Multisensory Motivational Center can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

**Do not submerge** the unit, as it will damage the electrical components.

**Do not use abrasive cleaners,** as they will scratch the surface of the unit.