

---

## Textured Roller Switch

---

### Feels great!

Once the user feels the bumps on this roller switch he/she will want to roll it again and again. It provides a very special somatosensory experience. Our roller switch is available as a switch only—spin the drum and the device is activated—or as a switch-toy, spin the drum and it turns on lights, vibration and music. Size: 11"L x 4½"W. Requires 4 AA Batteries. Weight: 1¾ lbs.

### Operation:

1. The Textured Roller Switch requires four AA batteries (Not included). The battery compartments are located under the base of the unit. Carefully turn the unit over, and then remove the Phillips head screws from the battery covers. Install new batteries, being careful to observe proper battery polarity. Alkaline batteries are recommended because they last longer.
2. Rotate or spin the Textured Roller and you will be rewarded with lights, music and vibration. Stop rotating or spinning of the Textured Roller will cause the Lights, Music and Vibration to stop.
3. To use the Textured Roller as a switch plug one end of the supplied jack cord into the Textured Roller's female jack and the other end into your adapted toy/device. Rotate or spin the textured Roller and your toy/device will activate as long as rotating or spinning of the Roller is sustained. Please note when used as a switch the Textured Roller's Lights, Vibration, Music will be deactivated.

### Troubleshooting:

**Problem:** The Textured Roller Switch has no light or Vibration.

**Action #1:** Check the batteries on the left side for strength and for proper battery polarity. Replace if weak or dead.

**Problem:** The Textured Roller Switch has no music.

**Action:** Check the two Batteries on the right side for strength and for proper battery polarity. Replace if weak or dead.

**Problem:** The Textured Roller Switch does not work at all.

**Action:** Check all Batteries for strength and for proper battery polarity. Replace if weak or dead.

**Problem:** Textured Roller Switch does not activate a toy or device.

**Action # 1:** Make sure all connections are tight.

**Action # 2:** Check batteries in toy/device, replace if weak or dead.

**Action # 3:** If adapters are being used they must be Mono, stereo adapters will not work.

### Care of Unit:

The Textured Roller Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

**Do not submerge** the unit, as it will damage the contents of the top and the electrical components.

---

# **Textured Roller Switch # 639**

**With Music, Lights & Vibration**

---

## **USER'S GUIDE**



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)