

---

---

## Twirling Bead Chain #6470

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697  
[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### Feel the cool beads!

Activate the built-in red gumball switch and the carousel begins to turn causing the colorful cool beads to gently touch your hand as they spin by. In addition, upbeat music plays and red and blue lights sparkle. Can also be activated with your own capability switch. A great sensory experience that also teaches cause and effect. *Size: 8½"Diameter x 11"H. Requires 2 C and 6 AAA Batteries. Weight: 3 lbs.*

1. Twirling Bead Chain requires 6 AAA & 2 C batteries (not included). Alkaline batteries must only be used (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. The battery compartments are located under the unit's twirling top and base. Gently turn the unit over to reveal 3 AAA black battery compartments. Pull back the Velcro straps and slid the covers off. When installing new batteries, always be careful to observe battery polarity. Replace battery covers and secure with straps. Next remove the small screw from the C battery compartment under the base, using a Phillips head screwdriver, and then slide the battery compartment cover off. Install C batteries. .Replace battery cover and screw. Never mix old and new batteries together or different brands or types together.
2. Turn the ON/OFF/VOLUME knob under the Twirling Top clockwise, you will hear an audible click. Adjust to knob for desired volume level. Also push the ON/OFF power button under the base to ON.
3. Simply activate your capability switch or just press the 2 ½" red gumball to watch the white circus top begin to spin and the colorful beads fly around in a wonderful circle. As the beads are swirling around you will see red and blues spots of light circling the base of the carousel while listen to upbeat music in the background. The feel of the beads crossing your hands while the top spins is fantastic.

4. To use the Twirling Bead Chain with a switch, plug, any Capability switch into the switch jack located on the side of the Twirling Bead Chain's base. Make sure to plug your switch all the way in. There should be no gaps.
5. It's a great tactile experience for all users. It provides hours of pleasurable sensory input. If you use the Carousel Bead Chain in low light you will be rewarded with a magnificent sparkling colored light show.

### Troubleshooting:

**Problem:** Twirling Bead Chain does not light up or play music.

**Action #1:** Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

**Action #2:** Make sure nothing is blocking or restricting the movement of the builtin Gumball switch.

**Problem:** Twirling Bead Chain does not activate an external switch.

**Action # 1:** Make sure your switch is plugged in all the way. There should be no gaps.

**Action # 2:** Check your switch with another device you know is working.

### Care of the Unit:

Twirling Bead Chain can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.

**Do not submerge** the unit, as it will damage the electrical components.