

---

---

## Musical Merry-Go-Round #6471

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)  
1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway  
Hawthorne, NY 10532  
Tel. 914.747.3070 / Fax 914.747.3480  
Toll Free 800.832.8697  
[www.enablingdevices.com](http://www.enablingdevices.com)

### ***Prancing horses!***

Press and hold your capability switch and this delightful Merry-Go-Round rotates while the horses go round and round as calliope music plays and lights twinkle. Great for developing visual attention and teaching cause and effect. Size: 8½"D x 10"H. Requires 2 C and 2 AA Batteries. Weight: 3¾ lbs.

1. Musical Merry-Go-Round requires 2 AA & 2 C batteries (not included). Alkaline batteries must only be used (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. The battery compartments are located under the unit's base. Gently turn the unit over to reveal the black battery compartments. When installing new batteries, always be careful to observe battery polarity. Remove the small screws from the compartments using a Phillips head screwdriver, and then slide the battery compartment covers off. Install batteries. Replace battery covers and screws. Never mix old and new batteries together or different brands or types together.
2. Turn the ON/OFF/VOLUME knob under the base clockwise, you will hear an audible click. Adjust to knob for desired volume level.
3. Next plug, any Capability switch into the switch jack located on the side of the Musical Merry-Go-Round's base. Make sure to plug your switch all the way in. There should be no gaps.
4. Press and hold your capability switch and this delightful Merry-Go-Round rotates while the horses go round and round as calliope music plays and lights twinkle. Great for developing visual attention and teaching cause and effect.

### **Troubleshooting:**

**Problem:** Musical Merry-Go-Round does not light up or play music.

**Action #1:** Check the Batteries for strength and for proper battery polarity. Replace if weak or dead.

**Action #2:** Make sure nothing is blocking or restricting the movement of Musical Merry-Go-Round.

**Action # 3:** Make sure your switch is plugged in all the way. There should be no gaps.

**Action # 4:** Check your switch with another device you know is working.

### **Care of the Unit:**

Musical Merry-Go-Round can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

**Do not use** abrasive cleaners, as they will scratch the surface of the unit.

**Do not submerge** the unit, as it will damage the electrical components.