

---

---

## Glitter Roll Music Base/Switch #696

---

---

### USER'S GUIDE



**For Technical Support:**

Call our Technical Service Department  
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

[customer\\_support@enablingdevices.com](mailto:customer_support@enablingdevices.com)



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

[www.enablingdevices.com](http://www.enablingdevices.com)

### **Encourages reaching and swiping!**

This innovative switch provides hours of multi-sensory stimulation with its roller action and built-in music box. The clear, cylindrical roller contains a potpourri of sparkling shapes. Roll it to activate the music base. Can also be used as an external capability switch without music when connected to a device. Size: 14"L x 7"W x 5"H. Requires 2 AA Batteries. Weight: 1<sup>3</sup>/<sub>4</sub> lbs.

#### **Operation:**

1. The Glitter Roll Music Box/Switch requires two AA batteries. Alkaline batteries are recommended because they last longer. Do not use rechargeable batteries because they supply lower voltage and the unit may not perform as well. The battery compartment is located in the bottom of the unit. When installing new batteries, always be careful to observe battery polarity.
2. The Glitter Roll Music Box/Switch can be connected to any toy or device via the 1/8-inch jack, which is located to the left on the front of the unit. The Glitter Roll Music Box/Switch #696 comes with a 1/8- to 1/8-inch connector cord for activating toys or devices with 1/8-inch jacks. If you need to use a 1/4-to 1/8-inch adapter, it must be a mono adapter, not stereo.
3. The glitter cylinder must be turned continuously in order to activate the music or a toy/device. After the cylinder stops turning, the music or toy will continue to be activated for a couple of seconds, before shutting off.

#### **Troubleshooting:**

**Problem:** Music becomes slow or distorted.

**Action:** Replace batteries.

**Problem:** The toy or device, which is connected to the Glitter Roll Music Box/Switch, does not activate.

**Action:** Replace batteries.

**Action:** Check or replace adaptor (if applicable).

**Problem:** Cylinder does not turn properly.

**Action:** Make sure the cylinder is set in the mold correctly and that nothing is caught between the cylinder and the mirror.

**Problem:** Cylinder squeaks or moves slowly.

**Action:** Place a drop of oil on the aluminum rod beside the end-caps.