
Grip & Puff Switch #700

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

This pneumatic switch is activated by slightly squeezing the vinyl grip. Can be used with our Grip Kit (see below). You can remove the Grip and use it as a Puff Switch, too. Comes with blue grip only. Weight: ½ lb. Requires 1 9-V Battery

Operation:

1. The Grip & Puff Switch does not require any batteries to operate.
2. Simply connect the Grip & Puff Switch to any switch adapted toy/device via the black cord included.
3. Now you are ready to go. Simply grasp and squeeze the blue vinyl grip to activate your toy/device.
4. This is a momentary switch. Therefore, you need to maintain pressure on the grip to keep the toy/device activated. Once you release the pressure, there will be a brief delay, and then the toy/device will turn off. If you would like to use the switch as a Puff switch there are a few more steps.
5. Remove the blue vinyl grip by separating the clear plastic tubing where the white plastic union joins it. Keep the plastic union with the blue vinyl grip
6. Attach the Anticontamination filter (the small disc) to the end of the clear plastic tubing. Next, attach the small piece of wider clear plastic tubing to the other end of the filter. Finally, attach the straw to the end.
7. Some extra tubing and another piece of “connector” tubing has been included if you want to extend the length of the clear plastic tubing.
8. Now place the straw in your mouth, close your lips around it, and puff moderately to activate your toy/device.
9. Again, this is a momentary switch. Therefore, you will need to maintain a constant puff on the straw to keep the toy/device activated. Once you stop puffing, there will be a brief delay; the toy/device will turn off.

Troubleshooting:

Problem: The unit fails to operate, or operates incorrectly.

Action #1: Make sure that all of the connections are tight (between tubes, unit box, adapter, connectors, etc.)

Action #2: Make sure that all of the connections are made in the correct order (See Operation: Step #8).

Action #3: Temporarily disconnect the clear tubing at the white connector to equalize the air pressure in the clear tubing, and then reconnect.

Action #4: With time and use, the straw and/or the Anticontamination filter will become clogged. It is best to change these components before they become 100% clogged, as this could damage the unit. See the back page of this manual for replacement part numbers. If possible, it is also a good idea to keep the switch box above the level of the user’s mouth. This limits the amount of saliva that enters the tubes.

Action #5: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Important Note:

This switch is extremely sensitive and may remain in the “ON” position when the atmospheric pressure changes. If this occurs, reset the switch by separating the clear tubing where it is joined together by the white plastic connector, and then reconnect.

Care of Unit:

The Grip & Puff Switch can be wiped clean with any household multi-purpose, non-abrasive cleaner and disinfectant.

Change your filter and straw regularly.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.