
Finger Isolation Button #716

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Teaches Pointing skills!

This recessed button encourages isolated touch and is designed to help your child gain the motor skills which are necessary for mastering computer keyboards and tablets. Size: 3" x 1½". Weight: 3 oz. It can also be used as a switch.

Operation:

1. The iPad Trainer Button does *not* require any batteries to operate.
2. Plug the iPad Trainer Button into the jack on your switch-adapted device.
4. Make sure that the connection between the iPad Trainer Button and your device is tight. There should be no gaps.
5. Simply press in the center of the iPad Trainer Button recessed button to activate your device.
6. The device will only remain activated while the iPad Trainer Button continues to be actuated. Once you release the button, the device will turn off.

Troubleshooting:

Problem: The iPad Trainer Button does not activate your device.

Action #1: Make sure that the connection between the iPad Trainer Button and your switch enabled device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #2: Try a different switch with your switch enabled device to rule out the device as the source of the problem.

Care of Unit:

The iPad Trainer Button can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.