
Mini Dome #72

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Increases visual attention!

Our best-selling Mini Dome has eye-catching colors and vibrations. Simply press anywhere on the dome or its plate, and the bright confetti spins, music plays and the dome vibrates. Now comes with on/off switch and volume control. Provides hours of visual, auditory and tactile stimulation. Size: 11"L x 9"W x 7"H. Requires 2 C Batteries. (No switch is required). Weight: 1¾ lbs.

Operation:

1. The Mini Dome requires two C size batteries Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located behind the dome. To remove the battery compartment cover, slide your fingers in the finger holes at the back of the unit and gently pull the cover upward. Install new batteries, being careful to observe proper battery polarity.
3. Replace the battery compartment cover by inserting one end of the cover into the end slot. Then, carefully bend the cover slightly upward in the center and slip the other end of the cover into the end slot.
4. Press anywhere on the dome or on the yellow plate to music, vibration and confetti.
5. The Mini Dome will continue playing while pressure is maintained on the dome or yellow plate. Once it is released, the Mini Dome will stop.

Troubleshooting:

Problem: The glitter does not spin.

Action #1: Occasionally pieces of glitter become wedged at the axis of the motor, preventing it from turning. If this appears to be the case, shake the unit gently in order to free the glitter.

Action #2: Replace the batteries paying close attention to proper polarity.

Problem: The music does not play or the unit does not vibrate.

Action #1: Replace the batteries paying close attention to proper polarity.

Care of Unit:

The Mini Dome can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.