

Easily activates several devices!

This joystick is perfect for teaching directionality as well as other spatial relationships. The joystick activates four different devices depending on the direction the switch is moved. Pushing the switch to the left will activate one device, moving it backwards will activate another and so forth. Size: 7"L x 6¼"W x 4½"H. Weight: ¾ lb.

Operation:

1. Connect four cords to external toys or devices via the four 1/8-inch jacks on the back of this switch. This unit is equipped with 1/8-inch jacks for use with capability switches with 1/8-inch connectors. If you need to use 1/4- to 1/8-inch adaptors, they must be mono adaptors, not stereo.
2. To operate a toy or device connected to the first jack, push the joystick to the left.
3. To operate a toy or device connected to the second jack, push the joystick forward.
4. To operate a toy or device connected to the third jack, pull the joystick back.
5. To operate a toy or device connected to the fourth jack, push the joystick to the right.

Care of Unit:

The Joystick can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.

Joystick #720

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com