
Textured Saucer Switches #722

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Saucer Switches

New textures!

Our popular saucer switches are now available with a new texture making it especially pleasing to all users. The slightest movement activates the device of your choice. Size: 7"D. Weight: 1¾ lbs.

Operation:

1. The Saucer Switch does *not* require any batteries to operate.
2. Plug the Saucer Switch into the jack on your switch-adapted toy/device.
3. If it is necessary for you to use an adapter, make sure that you use a MONO adapter, not a STEREO adapter.
4. Make sure that the connection between the Saucer Switch and your toy/device is tight. There should be no gaps.
5. Simply press anywhere on the Saucer Switch, to activate your toy/device.
6. The toy/device will only remain activated while the Saucer Switch continues to be actuated. Once you release the switch, the toy/device will turn off.

Troubleshooting:

Problem: The Saucer Switch does not activate your toy/device.

Action #1: Make sure that the Saucer Switch is on a flat surface (not tilted or vertical). This provides for optimal functioning.

Action #2: Make sure that the connection between the Saucer Switch and your toy/device is tight. There should be *no* gaps. This is a common error and an easy fix.

Action #3: Try a different switch with your toy/device to rule out the toy/device as the source of the problem.

Action #4: Try a different adapter (if applicable) to rule this out as the source of the problem.

Care of Unit:

The Saucer Switch can be wiped clean with any household multi-purpose cleaner and disinfectant.

Do not submerge the unit, as it will damage the contents and the electrical components.

Do not use abrasive cleaners, as they will scratch the surface of the unit.