
Lady Bug Switch #729
(With Lights, Music & Vibration)

USER'S GUIDE



For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)
1-800-832-8697
customer_support@enablingdevices.com



50 Broadway
Hawthorne, NY 10532
Tel. 914.747.3070 / Fax 914.747.3480
Toll Free 800.832.8697
www.enablingdevices.com

Cute and colorful!

Everyone is drawn to this colorful lady bug. The slightest touch rewards the user with a choice of lights, vibration and/or music. This item has a special control which allows the user to select any or all of the rewards. Size: 9"L x 6¼"W x 4"H. Requires 2 AA Batteries. Weight: 1 lb.

Operation:

1. The Lady Bug Switch requires two AA batteries (Not included). The battery compartment is located under the base of the unit. Carefully turn the switch over, and then remove the battery cover with a small Philips screwdriver. Install new batteries, being careful to observe proper battery polarity. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. Replace the battery case cover and secure it with the small screw. Do not over-tighten.
3. To turn on or off the desired stimuli (1-vibration; 2-music; 3-lights), push the small white rocker switches to the on or (open position). Open position means off. Please see Fig 1.

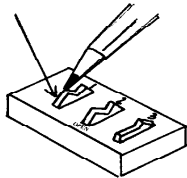


Figure 1.

4. Press anywhere on the Lady Bug's back and you will be rewarded with the selected stimuli vibration, lights or music.
5. To operate as a switch connect an external toy or device via the double ended male 1/8-inch to 1/8-inch cord , one end plugs into the 1/8-inch female jack on the side of the Lady Bug. Plug

the other end into your toy/device. If you need to use a ¼" adaptor, it must be a mono adaptor, not stereo. Press anywhere on the Lady Bug's back to activate your toy or device. Once you release pressure from the switch your toy or device will turn off.

6. When using as a switch you can turn On or Off any of the three stimuli.

Troubleshooting:

Problem: The Lady Bug Switch does not operate properly.

Action # 1: Make sure the batteries are in the battery compartment properly, and making good contact.

Action # 2: Make sure you have the desired stimuli selected as explained in operation No.3

Action # 3: If the unit still does not work properly, replace the batteries.

Problem: The Lady Bug Switch does not activate a connected toy/device.

Action #1: Make sure all connections are tight.

Action #2: Check batteries in toy/device, replace if weak or dead.

Care of Unit:

The Lady Bug Switch can be wiped clean with any household multi-purpose cleaner and disinfectant. We recommend Simple Green, which is a non toxic biodegradable all-purpose cleaner.

Do not submerge the unit, as it will damage the contents of the top and the electrical components.